

!!WARNING!!

THIS WATER BARRIER MUST BE USED!
QUINTRON IS NOT RESPONSIBLE FOR WATER
ENTERING AND CAUSING SEVERE DAMAGE TO
THE INSTRUMENT!

Instructions for use of Water Barrier with QuinTron SamplXtractor

To protect the Model SC from accidental infusion of water into the valve and column, we strongly recommend that a water barrier be attached to the SamplXtractor as shown below. The barrier will stop water from entering the instrument if water is accidentally pumped through due to overfilling the test tube! The barrier should be replaced if wet!

THIS WATER BARRIER MUST BE USED!

