



PRODUCT BULLETIN: 0004


BREATHTRACKER SOFTWARE UPDATE

IMPORTANT END-USER INFORMATION

PRODUCT CHANGE NOTIFICATION:

End-users are now being required to perform the half calibration procedure as a mandatory step during calibration. Many users are bypassing this procedure or not performing it correctly. This software update addresses this issue by not allowing the technician or end-user to bypass the half calibration procedure as well as other internal software fixes.

Below is the new display that will be seen.

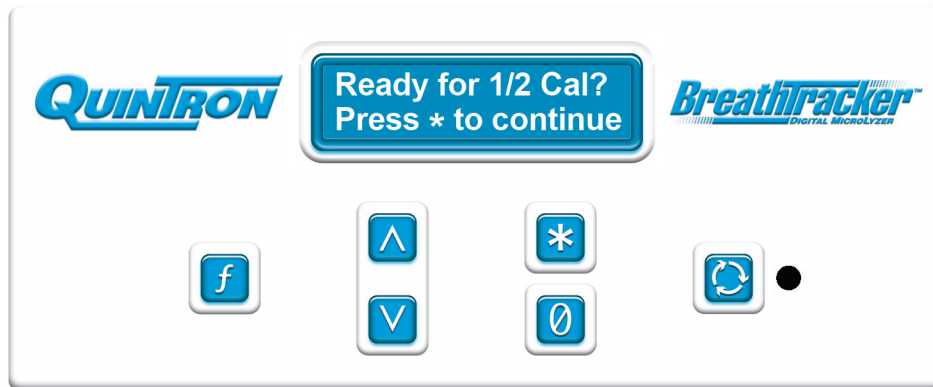

Users must perform this procedure to ensure proper analysis of patient samples.

Below is a list of the instruments models and the software version that includes this update.

<u>Model</u>	<u>Software</u>
BreathTracker SC	12
BreathTracker DP	10
BreathTracker H2+	10
BreathTracker H2	10

If you do not see the exact “Ready for 1/2 Cal?” display above during your calibration procedure, contact QuinTron to verify your software version at (800)542-4448 or (414)645-4222.

This update is mandatory for all BreathTracker users that do not have the latest software version installed.

To download a updated version of your instrument manual, please visit www.quintron-usa.com.

Make copies of this bulletin as desired to ensure
all users are properly notified of the change!