



## PRODUCT BULLETIN: 0007

### SIVRITE-4 TUBE (NEW PRODUCT)

REORDER #: QT01154-C

## IMPORTANT END-USER INFORMATION

### PRODUCT CHANGE NOTIFICATION:

We are pleased to announce improvements to our SivRite-4 product!

The desiccant mixture allows better humidity absorption and conditioning of room air, allowing for improved stability for your breath analyzer.

The bottom port has been eliminated; air flow now travels from the holes on the base of the tube.

Product replacement is easier to determine as the new mixture turns from blue to pink when expired.

Each order of SivRite-4 now contains two (2) of the new tube design. When the first tube is expired and the second tube is attached to the instrument, it's time to reorder!

*The part number (QT01154-C) and pricing of this product remains the same.*

### Instructions for Use:

1. Remove the bottom seal from the tube by pulling the red tab.
2. Remove the clear plastic luer cap from the top of the tube by gently twisting it.
3. Attach the tubing from the Air-In Port on your analyzer to the luer port.
4. Replace your SivRite-4 tube when the blue crystals have changed from **BLUE** to **PINK** up to the red line on the label.
  - After replacing your SivRite-4 tube, your instrument needs at least 2 hours to adjust before trying to calibrate or analyze patient samples.
  - REORDER SIVRITE-4 NOW.

**NOTE:** When expired, tube and cap are recyclable. The inner contents can be discarded into the trash.

PREVIOUS  
DESIGN



NEW  
DESIGN



Make copies of this bulletin as desired to ensure all users are properly notified of the change!